

HYDROSEAL

DIRECTIONS FOR USE:

1 Wash hands and prepare sight accordingly before applying **HYDROSEAL**.

2 Assess site and choose the appropriate sized sheet of **HYDROSEAL** to cover site. The adhesive border should extend beyond the dressing or area you want to prevent from getting wet.

Apply **HYDROSEAL** to clean dry skin. Creams and Lotions may inhibit the adhesive qualities of **HYDROSEAL**.

3 Remove top release liner from adhesive and adhere to skin.


4 Remove the bottom release liner adhesive and adhere to skin.

5 Remove release liner with *Performance Enhanced Tabs* and adhere to skin.

6 Remove opposite release liner with *Performance Enhanced Tabs* and adhere to skin.

7

Adhesive should be pinched together onto itself and around tubing to create a tight seal. Smooth outer border using firm pressure to ensure a proper seal around entire border of **HYDROSEAL**.

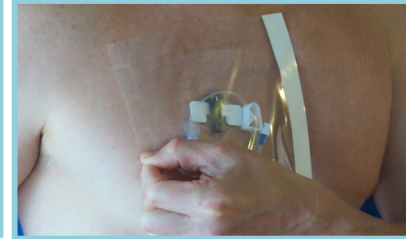

8

After showering, carefully remove **HYDROSEAL** in the direction of hair growth and dispose of accordingly.



HYDROSEAL

SIZES:

**5x5, 7x7, 9x9,
10x12, 12x14 and 7x22**

CAUTIONS:

- ◆ Discontinue use if patient has an adverse reaction to or irritation from the product.
- ◆ Do not remove and reposition adhesive once applied to skin.
- ◆ Routinely inspect **HYDROSEAL** during showering.
- ◆ Discard if packaging is opened or damaged.
- ◆ Single Use Only.
- ◆ Dispose of accordingly after use.
- ◆ Latex-Free.



26MEDICAL, LLC

Manufactured for:
2G Medical, LLC
80 Krif Rd., Unit #2